

Indian Council of Medical Research

Department of Health Research, Ministry of Health

and Family Welfare, Government of India

Date: 26.04.2021

Rapid Antigen Test Kits for COVID-19 (Oropharyngeal / Nasopharyngeal swabs)

Please Note:

- Below listed kits are validated with the mentioned batch number only. Responsibility for batch to batch consistency does not lies with ICMR.
- Minimum acceptance criteria of sensitivity and specificity of Rapid Ag Test Kits:
 - > Validated as a Point of Care Test (POCT) without transport to a laboratory setup-

Sensitivity: 50% and above; Specificity: 95% and above

> Validated in a laboratory setup with samples collected in Viral Transport Medium (VTM)-

Sensitivity: 70% and above; Specificity: 99% and above

• Antigen based rapid tests which are US-FDA approved can be used directly after due marketing approval from DCGI.

<u>Till date, 95 Antigen based Rapid Test Kits have been validated (including 17 revalidated</u> Kits), and the following are found to be satisfactory

S. No	Name of company	Name of the kit	Lot no. / Batch No.	Sample used for validation
1.	SD Biosensor, South Korea / India	STANDARD Q COVID-19 Ag	E055003	Nasopharyngeal swab
2.	LabCare Diagnostics Ltd., Valsad (Gujarat), India (Supplied by MyLab	Accucare COVID-19 Antigen Lateral Test Device	CVG200601 CVG200602 CVG200603	Oropharyngeal and Nasopharyngeal swabs
3.	Discovery Solutions) Trivitron Healthcare Pvt. Ltd., Chennai (TN), India	BIOCARD Pro COVID-19 Rapid Ag test kit	COVPGL-001 COVPGL-002 COVPGL-003	Nasopharyngeal swab
4.	Coris Bioconcept, Belgium	#COVID-19 Ag Respi Strip (VTM)	43242F2003 43512G2030 43464G2016	#Oropharyngeal swab in VTM
5.	Panion & BF Biotech., Taiwan	VSTRIP COVID-19 Antigen Rapid Test	IG10020S-R2004 IG10020S-R2005 IG10020S-R2006	Nasopharyngeal swab
6.	PCL Inc, South Korea	PCL COVID-19 Rapid FIA	2005K104 2005K105 2005K106	Nasopharyngeal swab
7.	Premier Medical Corporation, Valsad (Gujarat), India (Supplied by Cipla ltd.)	Sure status COVID-19 Ag Test (CIPtest COVID-19 Antigen Card Test)	9710120S 9710220S 9710320S	Nasopharyngeal swab



S. No	Name of company	Name of the kit	Lot no. / Batch No.	Sample used for
				validation
8.	Angstrom Biotech Pvt.	Angcard COVID-19 rapid	BCOVA03	Nasopharyngeal and
	Ltd., Alwar	Antigen Test kit	BCOVA04	Oropharyngeal
	(Rajasthan), India		BCOVA05	swabs
			BCOVA06	
9.	GenBody Inc.,		FMFC20201	Nasopharyngeal
	South Korea	GenBody COVID-19 Ag	FMFC16201	swab
		rapid Test kit (POCT)	FMFC02201	
10.	Ubio Biotechnology		SO64012010	Nasopharyngeal
	Systems Pvt. Ltd.,	SENSIT Rapid COVID-19 Ag	SO64012011	swab
	Kochin (Kerala), India	kit	SO64012012	
11.	Meril Diagnostics, Vapi		MRD131	Nasopharyngeal
	(Gujarat), India	COVID-19 Antigen	MRD132	swab
		Detection Test	MRD133	
12.	Alpine Biomedicals		LCOVG-010820	Nasopharyngeal
	Pvt. Ltd., Ambala	Alpine COVID-19 Antigen	LCOVG-020820	swab
	(Haryana), India	Rapid Test kit	LCOVG-030820	
13.	SD Biosensor, South	*Standard F COVID-19 Ag	FCO302010128	Nasopharyngeal
	Korea	FIA Test	FCO302010129	swab
		Analyser: STANDARD	FCO302009259	
		F2400		
14.	Oscar Medicare Pvt.	12100	D004	Nasopharyngeal
	Ltd., Delhi, India	Oscar CORONA Rapid Ag	D005	swab
	Lea., Berri, maia	Test kit	D006	3000
15.	ImmunoScience India	reserve	E146001	Nasopharyngeal
13.	Pvt. Ltd., Pune	ImmunoQuick COVID-19	E146002	swab
	(Maharshtra), India	Antigen Rapid Card test kit	E146003	31145
16.	STRUmed Solutions	, magair magair cara coormi	001	Nasopharyngeal
	Pvt. Ltd., Chennai	iNSTAXPLOR™ COVID-19	002	swab
	(TamilNadu), India	Ag – Rapid Antigen Test	003	0110.0
17.	Abbott Rapid	.gp.a	41ADF039A	Nasopharyngeal
	Diagnostics Division,		41ADF036A	swab
	Chicago	Panbio Covid Antigen	41ADF037A	
	2300	Rapid Test		
18.	ADVY Chemical Pvt.		01/1220	Nasopharyngeal
	Ltd., Thane		02/1220	swab
	(Maharashtra), India	EzDx COVID-19 Rapid Ag	03/1220	
	(manarasinia), mala	Test		
19.	Ortho Clinical	^{\$} Vitros SARS-CoV-2 Ag	0050	Nasopharyngeal
	Diagnostics, Mumbai	Test CLIA Kit	0051	swab in VTM
	(Maharashtra), India	Vitros 3600	0052	



S. No	Name of company	Name of the kit	Lot no. / Batch No.	Sample used for validation
20.	Sri Sathya Sai Institute		LORCAG1120A	Nasopharyngeal
	of Higher Learning,		LORCAG1120B	swab
	Anantpur (Andhra		LORCAG1120C	
	Pradesh), India	SAIC-19 Ag Kit		
21.	Medzome Lifesciences		MZCDA00120	Nasopharyngeal
	Pvt. Ltd., Solan		MZCDA00220	swab
	(Himachal Pradesh),	FutureCare COVID-19 Ag	MZCDA00320B	
	India	detection Test kit		
22.	Pathkits Healthcare		RK/CO/AG/11/20/01	Nasopharyngeal
	Pvt. Ltd., Gurugram	Pathkits SIMPLE COVID-19	RK/CO/AG/11/20/02	swab
	(Haryana), India	Ag Rapid Test	RK/CO/AG/11/20/03	
23.	Biofootprints		MT057001	Nasopharyngeal
	Healthcare Pvt. Ltd.		MT057002	swab
		MyTest COVID-19 Ag Test	MT057003	
24.	Zephyr Biomedicals		ZRD/20/K-64	Nasopharyngeal
	(Tulip Diagnostics)	CoviRAT COVID-19 Rapid	ZRD/20/K-65	swab
	Goa, India	Antigen Test kit	ZRD/20/K-66	
25.	Meril Diagnostics Pvt.	Covid-19 Antigen	MRD151	Nasopharyngeal
	Ltd., Vapi (Gujrat),	Detection Test (FIA)	MRD152	swab
	India	Immunofluorescence	MRD153	
		Analyzer CHF200		
26.	Sidak Lifecare Pvt.	One Step novel corona	SLC_COAG_21/20	Nasopharyngeal
	Ltd., Jhajjar (Hayana),	virus (COVID-19) Antigen	SLC_COAG_22/20	swab
	India	Test Kit	SLC_COAG_23/20	
27.	Diagnocure (INDIA),		XCVG501	Nasopharyngeal
	Solan, HP	Xamin Covid-19 Ag Rapid	XCVG502	swab
		Test Device	XCVG503	
28.	Dia Sure		CAG-E-001	Nasopharyngeal
	Immunodiagnostics	DSI COVID-19 Ag RAPID	CAG-E-002	swab
	LLP, Delhi, India	Test Kit	CAG-E-003	
29.	Biolab Diagnostics		9127T	Nasopharyngeal
	India Pvt. Ltd., Mumbai		9120T	swab
	(Maharashtra), India	Rapid Covid 19 Ag kit	9133T	
30.	NextGen In Vitro	COVSCAN SARS-Co V-2	CVRA T022 1-01	Nasopharyngeal
30.	Diagnostics (P.)	Antigen Detection kit	CVRA T022 1-01 CVRA T022 1-02	swab
	Limited., Faridabad	Thingen Detection kit	CVRA T022 1-03	SWaD
	(Haryana), India			
31.	Oscar Medicare Pvt.		CAG-H0004	Nasopharyngeal
01.	Ltd., Haridwar		CAG-H0005	swab
	(Uttarakhand), India	COVID-19 Antigen Card	CAG-H0006	
	() 5555 555 555 556	Test		



Indian Council of Medical Research

Department of Health Research, Ministry of Health
and Family Welfare, Government of India

S. No	Name of company	Name of the kit	Lot no. / Batch No.	Sample used for
				validation
32.	Agappe Diagnostic	AG-Q COVID-19 N	R003001	Nasopharyngeal
	Pvt. Ltd., Kerala	Antigen Rapid Diagnostic	R003002	swab
		Kit	R003003	
33.	Abbott Rapid	PanBio COVID-19	41ADF315A	Nasal Swab
	Diagnostics Division,	Antigen Rapid Test	41ADF316A	
	Chicago	Device	41ADF317A	
34.	Athenese-Dx Private		RD1690C	Nasopharyngeal
	Limited, Chennai	TRUSTline COVID-19 Ag	RD1691C	swab
		Rapid Test	RD1692C	
35.	TaiDoc Technology		20J005-0000	Nasopharyngeal
	Corporation, Taiwan	FORA® COVID-19	20J122-0000	swab
		Antigen Rapid Test	20J129-0000	

^{\$}Guidance for use is placed at **Annexure I**

List of Rapid Ag Test kits validated and not approved is placed at **Annexure IV** Individual validation reports of the above listed kits can be shared with the State Governments on request. Request may be sent at drneetu.vijay@icmr.gov.in

^{*}Guidance for use is placed at **Annexure II**

[#]Guidance for use is placed at **Annexure III**



Indian Council of Medical Research

Department of Health Research, Ministry of Health
and Family Welfare, Government of India

Annexure I

Guidance for use of VITROS® SARS CoV2 Antigen CLIA based Test from Ortho Clinical Diagnostics

- 1. The VITROS® SARS CoV2 Antigen assay is a chemiluminescent immunoassay intended for the qualitative detection of SARS-CoV-2 nucleocapsid antigens in nasopharyngeal (NP) specimens from individuals who are suspected of COVID-19 within one to five days of the onset of symptoms, or mid-turbinate specimens collected from asymptomatic individuals.
- 2. The assay should be performed in VITROS® 3600 Immunodiagnostics system or VITROS® 5600 / VITROS® XT 7600 Integrated system from Ortho.

VITROS SARS CoV2 antigen assay - Procedural Steps: -

Stage 1: Nasopharyngeal swab specimen collection:

- 1. Collect a nasopharyngeal swab specimen by inserting the sterile swab into the nostril.
- 2. Push the sterile swab until resistance is met at the level of the turbinate.
- 3. Rotate the sterile swab several times against the nasopharyngeal wall & leave in the place for 10 seconds to saturate the swab tip.
- 4. Remove the swab from the nostril carefully.
- 5. Place the swab specimen into the viral transport medium buffer tube and close the tube tightly.
- 6. Transport the swab sample in VTM to the laboratory in cold chain.
- 7. The sample can be stored in the Room temperature (Below 30°C) up to 24 hrs from the time of sample collection or at 2 8°C for up to 48 hrs from the time of sample collection.

Stage 2: Sample preparation for testing:

- 1. Sample preparation needs to be performed in BSL-2 level cabinet in the Laboratory.
- 2. Mix the swab specimen in VTM tube well (vortex approximately 3-5 seconds).
- 3. Transfer 100 µL VITROS® SARS-CoV-2 Antigen Extraction Buffer into a labelled new sample tube.
- 4. Add 400 μL viral sample to the above tube (to maintain 1:4 ratio of extraction buffer: sample)
- 5. Mix well (Cap/Plug the sample tube and vortex approximately 3-5 seconds)

Stage 3: Sample processing in VITROS® systems:

- 1. Place/load the prepared/extracted sample tube in stage 2, after de-capping on to the VITROS® instrument V3600/V5600/XT7600; An amount of 80 μ L of extracted sample is used for each determination.
- 2. Program VITROS 3600 / VITROS 5600 / VITROS XT 7600 system to process the samples for CV2Ag. The system can used to program test either in a STAT/Random/Batch mode.
- 3. System processes the samples automatically using disposable VersaTips for both sample as well as reagents to prevent any cross-contamination. The results will be delivered in 48 minutes after sample aspiration, in the form of S/Co value.



Indian Council of Medical Research

Department of Health Research, Ministry of Health
and Family Welfare, Government of India

Record and interpret the results as follows:

Results are automatically calculated by the VITROS Immunodiagnostic and VITROS Integrated Systems.

Result = $\frac{Signal \ of \ test sample}{Signal \ of \ Cutoff \ (Cutoff \ Value)}$

Interpretation of Results: Sample results will be displayed with a numerical signal to cutoff (S/C) value and a "Non-reactive" (negative) or "Reactive" (positive) label.

Results (S/C)	< 1.00	≥ 1.00
Result Text	Non-reactive (negative)	Reactive (positive)

Signal to cutoff numerical values will increase as the amount of SARS-CoV-2 antigen present in the sample increases.

General Guidance:

- It is recommended that the test should be performed as per latest ICMR guidelines for COVID-19 testing and manufacturer's IFU (instructions for use)
- Samples may be stored at ≤-20 °C and may be subjected to 5 freeze-thaw cycles.
- VITROS® SARS CoV2 antigen assay is to be used for the qualitative detection of SARS-CoV-2 antigens from nasopharyngeal or mid-turbinate swab specimens only. The nasal and saliva samples are not yet validated.
- Positive results indicate the presence of viral antigens, but clinical correlation with patient history and other diagnostic information is necessary to determine infection status. The test can detect both viable and non-viable SARS CoV2 antigenic material.
- A false negative test result may occur if the level of antigen in a sample is below the detection limit of the test or if the sample was collected or transported improperly; therefore a negative test result does not eliminate the possibility of SARS CoV2 infection. Individuals with CT values greater than or equal to 34 are unlikely to have replication competent virus.
- Negative results from patients with symptom onset outside of one to five days of symptom onset should be treated as presumptive.
- The sensitivity/antigen detection ability of the VITROS® SARS CoV2 antigen assay may vary depending upon the quality of the VTM buffer used for sample collection. The five validated VTM are − 1. CDC formulation based VTM, 2. COPAN (BD) UTM, 3. NewProv VTM, 4. Hardy VTM, & 5. Remel M4RT VTM. Labs using other VTM/UTM are directed to validate the compatibility internally with the VITROS® CLIA antigen assay, before delivering patient results.



Indian Council of Medical Research

Department of Health Research, Ministry of Health

and Family Welfare, Government of India

Annexure II

Guidance for use of Standard F covid-19 Ag FIA Test (SD Biosensor)

Brief SOP for the Standard F COVID-19 Ag Test:

- 1. STANDARD F COVID-19 Ag FIA is a Europium based fluorescent immunoassay for the qualitative detection of the specific nucleocapsid protein antigen from SARS- CoV-2 in nasopharyngeal swab specimen. STANDARD F COVID-19 Ag FIA should be used with Standard F analysers (F100, F200, F2400) manufactured by SD Biosensor.
- 2. The Kit Contents are the Test Device, Specimen Extraction Buffer Tube, Sterile Swab for sample collection, Nozzle Cap & Instructions for Use

Standard F COVID-19 Ag FIA Procedural Steps: - Stage 1: preparing the specimen

- 1. To collect a nasopharyngeal swab specimen, insert the sterile swab into the nostril
- 2. Using the gentle rotation, push the sterile swab until resistance is met at the level of the turbinate
- 3. Rotate the sterile swab several times against the nasopharyngeal wall & leave in the place for 10 seconds to saturate the swab tip
- 4. Remove the swab from the nostril carefully
- 5. Repeat the above procedure in the other nostril
- 6. Place the swab specimen into the buffer tube. While squeezing the buffer tube, stir the swab more than 10 times. This buffer inactivates the virus thereby reducing the biosafety & biosecurity requirements.
- 7. Remove the swab while squeezing the sides of the tube to extract the liquid from the swab
- 8. Press the nozzle cap tightly onto the buffer tube

Stage 2.1: Performing the Test (READ ONLY MODE)

- 1. Prepare the test devices depending on the workload
- 2. Prepare Extracted Specimens in the buffer tubes
- 3. Mark the Test Cartridges as per specimen application plan (from 1, 2, 3 ... and patient ID)
- 4. Apply 4 drops of extracted specimen to the specimen well of the test device as per above sequence at about 20 seconds intervals
- 5. Leave the test device for 15 minutes on a flat surface for incubation
- 6. Prepare F100 or F200 analyser & select the READ ONLY MODE as per user manual
- 7. Insert the test device into the analyser which has completed incubation duration
- 8. Select the specimen type
- 9. The analyser will automatically scan & display the results in 1 minute after specimen type selection.



Indian Council of Medical Research

Department of Health Research, Ministry of Health
and Family Welfare, Government of India

Stage 2.2: Performing the Test (STANDARD MODE)

- 1. Choose Standard Test Mode & Insert the Test Device when prompted
- 2. Apply 4 drops of extracted specimen to the Specimen well of the test deice
- 3. After applying the specimen, immediately press 'TEST START' button
- 4. The analyser will automatically display the result after 15 minutes

General guidance:

- Specimen may be stored at room temperature for up to 30 minutes 1 hour in the buffer tube
- Result Time for COVID-19 Ag FIA on Standard F analyzer system under READ ONLY MODE is 1
 minute only after completing an incubation for 15 minutes
- Sample should be collected from both the nostrils
- Print out can be taken within 10 seconds after getting the result on the analyzer screen
- Patient ID can be written on the test cartridges for record keeping
- Print Results could be used further for reporting to ICMR or State Governments as per their statutory requirement
- Memory 5,000 results in F2400, 3,000 results in F200 & 1,000 results in F100 could be stored for later reference & analysis
- Data Transfer The data stored in the analyzers could be transferred over LIS & HIS interfaces in F2400 & F200 analyzers and could be made available easily for clinical decision making
- It is recommended that the test should be performed onsite under strict medical supervision, following proper COVID-19 testing guidelines & maintaining the kit temperature between 2° to 30° C.



Indian Council of Medical Research

Department of Health Research, Ministry of Health
and Family Welfare, Government of India

Annexure III

Guidance for use of COVID-19 Ag Respi-Strip (CorisBioConcept)

*COVID-19 Ag Respi-Strip (Coris BioConcept) involves a different methodology of testing as compared to the other two antigen testing kits approved by ICMR till now. This test cannot be performed bedside and requires a BSL-2 set-up for running the test.

Brief methodology of use of COVID-19 Ag Respi-Strip (CorisBioConcept):

- 1. The Nasopharyngeal and/or Oropharyngeal swab will be collected from COVID-19 suspect patient in Viral Transport Medium (VTM).
- 2. The collected swab in VTM will be brought to the laboratory in appropriate cold chain conditions.
- 3. Once the sample is brought to the laboratory, it will need to be handled in a BSL-2 level cabinet for aliquoting, putting in lysis buffer and loading the test strip.
 - Steps 1 & 2 will be performed as per the standard practice followed for collection and transport of samples for COVID-19 RT-PCR test.
 - Step 3 will need to be performed as per manufacturers' instructions given with the test kit.

Differences between COVID-19 Ag Respi-Strip (CorisBioConcept) and other antigen test kits approved by ICMR are as follows:

approvea by ICMR are as Jollows:				
COVID-19 Ag Respi-Strip (Coris BioConcept)	Other approved Ag Assays			
Cannot be employed as a point of care test	Can be employed as a point of care test			
Test kit does not have a sample collection swab	Test kit has a sample collection swab			
Nasopharyngeal and/or Oropharyngeal swab in VTM should be used.	STANDARD Q COVID-19 Ag kit and BIOCARD Pro COVID- 19: Only nasopharyngeal swab should be used. For COVID-19 Antigen Lateral test device: Throat/nasal/nasopharyngeal swab can be used.			
Sample has to be collected using the standard swab provided with VTM. Once collected, the sample needs to be put into the VTM tube.				
The collected sample needs to be transported into a BSL-2 lab in cold chain conditions	No transport to a BSL-2 lab is required as this is a point of care test.			
100 μ l of the VTM sample needs to be added to the dilution buffer in a test tube provided with the kit.	The extracted sample should be shaken and 2-3 drops to be added to the well of the lateral flow strip.			



Read results in 15 mins or earlier after insertion	For STANDARD Q COVID-19 Ag: Results should be read
of strip into tube containing sample and dilution	between 15-30 mins.
buffer. The strip should be discarded after 15	For COVID-19 Antigen Lateral test device: results should
minutes.	be read within 15-20 minutes.
Control line may not appear in a positive test.	Control line must appear for the test to be valid.
Storage temperature: 4-30°C	Storage temperature: 2-30°C



Indian Council of Medical Research

Department of Health Research, Ministry of Health
and Family Welfare, Government of India

Annexure IV

List of Rapid Ag Test kits validated, and Not Approved

S. No	Name of Company	Name of Kit	India/ Other countries	Name of the supplier
1.	Bhat Bio-Tech India, Bangalore	COVID-19 Antigen Rapid Card Test	India	Invex Health, Mumbai India
2.	POCT Services Pvt Ltd.	Q-Line Rapid COVID-19 Rapid Antigen Test	India	POCT Services Pvt Ltd
3.	HLL Lifecare Ltd	Makesure Covid-19 Antigen Rapid Card	India	HLL Lifecare Ltd
4.	Formosa Biomedical Technology Corp.	Formosa One Sure SARS- COV-2 Ag rapid Test Kit	Taiwan	Intai Lifesciences LLP, India
5.	Rapigen Inc	Biocredit COVID-19 Ag	South Korea	Imperial Life Sciences, Gurgaon
6.	Camtech Diagnostics	Camtech COVID-19 rapid Antigen Test	Singapore	Althea Pharma Pvt. Ltd, Mumbai
7.	M/s Medsource Ozone Biomedicals Pvt Ltd.	COVID-19 Antigen Rapid Test	India	M/s Medsource Ozone Biomedicals Pvt Ltd.
8.	Aspen Laboratories Ltd.	ASPEN COVID antigen rapid test	India	Aspen Laboratories Ltd., Delhi
9.	Genuine Biosystem Private Limited	GB QUIK Covid-19 Rapid Antigen test Kit	India	Genuine Biosystem Private Limited
10.	GenBody Inc.	GenBody COVID-19 Ag rapid Test kit (VTM)	South Korea	Vishat Diagnostics Pvt. Ltd, Mumbai
11.	ScheBo BioTech Netanyastr.2	ScheBo SARS-CoV-2 Antigen ELISA	Germany	GastroLab India Pvt. Ltd.
12.	ManKind Pharma	Rapid Point of Care COVID- 19 Ag detection kit	India	ManKind Pharma
13.	Seloi Healthcare Pvt. Ltd.	Insta COVID-19 Ag one step SARS-CoV2 rapid test	India	Seloi Healthcare Pvt. Ltd.
14.	Genes2Me	VIRALSCREEN COVID-19 Rapid Antigen Test Kit	India	Genes2Me
15.	ImmunoScience India Pvt. Ltd.	Immuno Quick COVID-19 Ag Test Kit (Dipstick)	India	ImmunoScience India Pvt. Ltd.
16.	J. Mitra & Co. Pvt. Ltd., Bangalore	COVID-19 Antigen Dot Test	India	J. Mitra & Co. Pvt. Ltd., Bangalore
17.	Shanghai Liangrun Biomedical Technology Co. Ltd, China	LionRun SARS-CoV 2 Ag	China	Shanghai Liangrun Biomedical Technology Co. Ltd, China
18.	Voxtur Bio Ltd., India	VOXPRESS COVID-19 Ag	India	Voxtur Bio Ltd., India



S. No	Name of Company	Name of Kit	India/ Other countries	Name of the supplier
		rapid test		
19.	Genomic Diagnostics, USA	COVID-19 viral Ag Test kit	USA	Genomic Diagnostics, USA
20.	Salofa Oy	SARS-CoV-2 Antigen quantitative assay kit (Enzyme-linked immunoassay)	Finland	MODILIFE Kesha Sales Private Limited, India
21.	TaiDoc Tech Coorporation, Taiwan	V TRUST Covid-19 Antigen Rapid Test	Taiwan	UR DISTREE PVT LTD., India
22.	M/s Metadesign Solutions Pvt. Ltd., Gurugram (Haryana), India	PK COVID-19 Ag Rapid Detection kit	India	M/s Metadesign Solutions Pvt. Ltd., Gurugram (Haryana), India
23.	Mediforce Healthcare Pvt. Ltd., Merrut (Uttar Pradesh), India	MediCheck COVID-19 Antigen Test	India	Mediforce Healthcare Pvt. Ltd., Merrut (Uttar Pradesh), India
24.	VanGuard Diagnostics Pvt. Ltd., Delhi, India	VDx COVID-19 Rapid Antigen Test	India	VanGuard Diagnostics Pvt. Ltd., Delhi, India
25.	Corios Bioconcept, Belgium	COVID-19 Ag Respi-Striip (POCT)	Belgium	Vishat Diagnostics Pvt. Ltd, Mumbai
26.	NDFOS Co. Ltd., Seoul, Korea	ND COVID-19 Ag Test Strip	Korea	Life Technologies (India) Pvt. Ltd.
27.	Bioline Diagnostics LLP, Delhi, India	SARS-CoV2 Antigen Lateral Flow Assay	India	Bioline Diagnostics LLP, Delhi, India
28.	Kilpest India Ltd., Bhopal (Madhya Pradesh), India	TRURAPID COVID-19 Ag Test	India	Kilpest India Ltd., Bhopal (Madhya Pradesh), India
29.	Calth Inc., Republic of Korea	LabGun COVID-19 Rapid Ag kit	Republic of Korea	Siemens Healthcare Pvt. Ltd., India
30.	Humasis Co. Ltd., South Korea	Humasis COVID-19 Ag Test	S Korea	MT Promedt Consulting, Germany
31.	Karwa Enterprises Pvt. Ltd., India	COVID-19 Antigen Rapid Test Cassette	India	Karwa Enterprises Pvt. Ltd.
32.	J. Mitra & Co. Pvt. Ltd.,	COVID-19 Ag Card Test	India	J. Mitra & Co. Pvt. Ltd.,



S. No	Name of Company	Name of Kit	India/ Other	Name of the supplier
			countries	
	Bangalore			Bangalore
33.	M/S Biocan Diagnostics	Tell Me FAST COVID-19 Ag	Canada	M/S Biocan Diagnostics
	Inc., Canada	Test		Inc., Canada
34.	MyLab Discovery	MyLab PathoCatch SARS-	India	MyLab Discovery
	Solutions	CoV-2 Ag FIA test kit		Solutions
35.	Mediclone Biotech Pvt.	@sight COVID-19 Antigen	India	Mediclone Biotech Pvt.
	Ltd, Chennai	Test kit		Ltd, Chennai
36.	Capital Health Services		India	Capital Health Services
	India Private Limited,	Capital IHF COVIDAG 2019-		India Private Limited,
	Hyderabad	nCOV RBD		Hyderabad
37.	Dynamed Equipments,		India	Dynamed Equipments,
	<u>Chennai</u>	C19 Antigen Test Kit		<u>Chennai</u>
38.	Pentavalent BioSciences		India	<u>Pentavalent</u>
	Pvt. Ltd., Bengaluru,	Pentascan COVID-19 Ag		BioSciences Pvt. Ltd.,
	<mark>Karnataka</mark>	Rapid Test kit		Bengaluru, Karnataka
39.	Oscar Medicare Pvt.	Oskit Corona Antigen FIA-R	India	Oscar Medicare Pvt.
	Ltd., Delhi, India	Test		Ltd., Delhi, India
40.	Avecon Healthcare Pvt.	MAXLINE COVID-19	India	Avecon Heathcare Pvt.
	Ltd.	Antigen Test		Ltd.
41.	Edge Pharma Pvt. Ltd.	EdgeXpress COVID-19	India	Edge Pharma Pvt. Ltd.
		Antigen Detection Test		
42.	Karwa Enterprises Pvt.	SARS-COV 2 Rapid Antigen	India	Karwa Enterprises Pvt.
	Ltd., India	Test (N protein)		Ltd., India
43.	Bioline Diagnostics LLP,	FIRSTVIEW COVID-19 Ag	India	Bioline Diagnostics LLP,
	<mark>New Delhi</mark>	FIA Test Device		<mark>New Delhi</mark>